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EN : Electric baby nasal aspirator

A baby often has a blocked or runny nose but until the age of three years, she cannot blow his nose on his own The electric baby nasal aspirator effectively clears secretions and nasal mucus blocking baby's nose and making breathing difficult (difficulty sleeping, trouble feeding...).

Do not use appliance for anything else. Do not use baby nasal aspirator if nasal passages are irritated. In the event of an infection, it is recommended that a doctor be consulted. Please take note of these instructions before using and keep them in a safe place for later reference.

Please read these instructions carefully before use and keep them in a safe place for future reference.

DETAILED DIAGRAM OF APPLIANCE (figure 1)

- 1 - Reusable tips (x3)
- 2 - Reservoir
- 3 - ON button
- 4 - Battery compartment

INSERTING BATTERIES (figure 2)

Open battery compartment by unscrewing the screw. Insert two AA 1.5V alka- line batteries into the intended chamber, taking into account the indicated pola- rity. Close the lid again and lock.

WARNING REGARDING BATTERIES

- Keep these instructions because they contain important information.
- Do not mix used and new batteries.
- Do not mix alkaline batteries, standard (carbon-zinc) batteries and rechar- geable (nickel-cadmium) batteries.
- Do not recharge non-rechargeable batteries.
- Before putting batteries in ensure that all contact surfaces are clean.
- Remove batteries when the device is not being used for long periods of time.
- When the sound of the pump and/ or pump suction weakens, the batteries must be replaced.
- Used batteries must be taken to an appropriate collection place.

USING THE BABY NASAL ASPIRATOR (figure 3)

Gently place the tip at the entrance of the congested nasal passage, without entering the nose. Press the ON button to start the suction action. Only a few seconds are needed to remove the secretions (maximum 5 seconds). For maximum effectiveness, the nose should be tilted to the bottom and the appliance should form a 45° angle. Do not use the nasal aspirator when the child is lying down. ON button towards the top. After usage, the collected nasal secretions remain in the reservoir. Empty and clean reservoir and tip between each use.

USAGE INSTRUCTIONS

For greater effectiveness of the product, it is best to block the opposing nostril during aspiration. Use a saline solution along with the baby nasal aspirator to help clear the secretions. While aspirating, ensure the baby's mouth is open to ease inner ear pressure. Take care not to suction the nasal wall.

CLEANING AND MAINTENANCE (figure 4)

DISMANTLING:

The nozzle and reservoir come in one piece which is easy to detach from the aspirator so that it can be cleaned thoroughly after each use. Pull upwards to remove it from the nasal aspirator, ensuring that you do not twist it as you do so.

CLEANING THE TIP:

Clean in dishwasher, either in the cutlery basket, or in the top rack. Or clean by hand with soapy water.

CLEANING THE RESERVOIR:

Clean in dishwasher, either in the cutlery basket, or in the top rack. Or clean by hand with soapy water.

CLEANING THE BODY OF THE NASAL ASPIRATOR:

Never place body of the appliance under water or in any other liquid, it is not water-resistant. Use a soft, dry cloth to clean the main unit. Do not clean with solvents or abrasive products.

ASSEMBLAGED:

When attaching the reservoir to the aspirator, you will hear a "click" when it is correctly in place and the marker is no longer visible.

IMPORTANT RECOMMENDATIONS

Before usage, check the appliance is correctly assembled. Do not push the nasal tip of the aspirator deeply into the baby's nostril. Do not submerge the part of the appliance containing the batteries in any liquid whatsoever. Clean appliance after each use. Do not leave appliance in reach of children. Keep product in a dry environment at ambient temperatures. Remove batteries if appliance remains unused for long periods.

TECHNICAL FEATURES

Suction power: 430mm/Hg.
Power: two 1.5V AA batteries.
Weight: 150 g.
Dimensions: 170x60 mm
Suction tip: supple silicon that is washable and reusable.
Usage requirements: between 15°C and 35°C.

GUARANTEE

Electric baby nasal aspirator is guaranteed for two years from date of purchase, subject to normal usage. Dorel France cannot be held responsible for possible damage caused by:
• usage not in accordance with instructions
• unsuitable, incorrect or unreasonable usage
• repairs or modifications undertaken by unauthorised technicians.

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