4

B BREASTMILK STORAGE BOTTLES

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BREASTFEEDING EXPERTS FOR OVER **30** YEARS

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GB Lansinoh® Breastmilk storage bottles

Congratulations on your decision to give your baby the best possible start in life by breastfeeding. The multitude of health benefits for both you and your baby are well documented. Although breastmilk is always best, providing breastmilk for your baby when you are to be away isn't always easy. It requires a special effort, and you should be very proud of your commitment to provide your baby with the perfect nutrition for ideal growth and development.

is due to the breakdown of milk fats. The milk is safe and most babies will still drink it. If the milk smells sour do not use.

DO NOT MICROWAVE HUMAN MILK! Microwaving alters the composition of the milk. Microwaving may also create "hot spots" in the milk that could burn your baby.

DO NOT thaw breastmilk in very hot or boiling water. Once breastfeeding has been established, if you choose to use a feeding bottle to feed your expressed breastmilk to your baby, then our Breastmilk Storage Bottle is compatible with the Lansinoh® mOmma® NaturalWave™ test which enables you to maintain established breastfeeding patterns, ensuring your baby will return to your breast.

Should you choose to use the Lansinoh® Breastmilk Storage Bottles with a teat, please take the following precautions:

For your child's safety and health:

- Before first use, clean the product.
- Prior to using your bottles for the first time, disassemble all parts and boil them for 5 minutes. This is to ensure hygiene. Be sure to have enough water so all parts are floating and are not touching the sides of the pot.
- Remove all parts from water immediately after boiling.
- Completely air dry before next use.
- After each use, disassemble and wash all parts in warm soapy water.
- Rinse in clean water and allow parts to air dry on a clean towel.
- Alternatively, the disassembled parts can be cleaned in the upper compartment of your dishwasher.

Bottle cleaning and sterilising

• Before first use, clean the product.

• Prior to using your bottles for the first time, disassemble all parts and boil them for 5 minutes. This is to ensure hygiene. Be sure to have enough water so all parts are floating and are not touching the sides of the pot.

• Remove all parts from water immediately after boiling.

• Completely air dry before next use.

• After each use, disassemble and wash all parts in warm soapy water.

• Rinse in clean water and allow parts to air dry on a clean towel.

• Alternatively, the disassembled parts can be cleaned in the upper compartment of your dishwasher.

Lansinoh® breastmilk storage bottles are ideal for collecting, storing and freezing your breastmilk. The ergonomic, wide-neck bottles fit with the Lansinoh® 2in1 Electric Breast Pump, the Lansinoh® Single Electric Breast Pump as well as the Lansinoh® Manual Breast Pump.

Storing and thawing your breastmilk

Breastmilk should be stored in a clean, sealed container.

How long to store breastmilk

Depending on the country, breastmilk storage guidelines may vary, therefore it is recommended that you check with your midwife, health visitor or breastfeeding counsellor for specific storage instructions. In the United Kingdom, The Breastfeeding Network recommends following the below guidelines when storing your breastmilk.

PLACE	MAXIMUM TIME
Fridge: 5 to 10°C	6 hours
Fridge: 0 to 4°C	3 days
Fridge: -18°C or lower	6 months

Previously frozen breast milk

Defrosted in fridge	12 hours
Defrosted outside fridge	Use immediately

September 2004 © The Breastfeeding Network

Caution

- Keep all components not in use out of the reach of children.
- Use of other sanitising methods (e.g., microwave) is not recommended and may damage the product.
- Do not clean, store or allow to come into contact with solvents or harsh chemicals. Damage could result.
- Do not store in use, store in a dry and covered place.
- **DO NOT** use your breastmilk in the fridge/ freezer door. The door tends to be the warmest part of the fridge/freezer. Date all milk and use the oldest milk first.

Thaw frozen milk in the refrigerator or place the container under cool running water, gradually warming the water until the milk is thawed. If defrosted in fridge use within 12 hours, if defrosted outside of the fridge use immediately.

DO NOT refreeze breastmilk once it has thawed. Gently swirl the bottle of thawed breastmilk before feeding to baby to mix the layers that have separated. Breastmilk is not homogenised and the "cream" will rise to the top of the bottle. Breastmilk can vary in colour and consistency depending on the time of the day that the milk was expressed, what you have eaten and the age of the baby at the time of pumping.

Sometimes thawed milk may smell or taste soapy. This

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