babymoov®

Garantie

À VIE*

LIFETIME
Warranty*

Babydoppler Connect

Ref. A062201

Notice d'utilisation (06.2018) • Instructions for use (06.2018) • Benutzungsanleitung (06.2018)
Gebruikshandleiding (06.2018) • Instrucciones de uso (06.2018)
Istruzioni per l'uso (06.2018)

Babymoov
Parc Industriel des Gravanches
16, rue Jacqueline Auriol
63051 Clermont Ferrand 2 - France
www.babymoov.com

Hotline (only for France):
04 73 28 37 37

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TABLE OF CONTENTS

01. Introduction
02. Copyright and trademark
03. Contents of the box
04. Manufacturer's warning and liability
05. Presentation of the device
06. Precautions for use
07. Technical features
08. Symbols
09. Usage
10. Maintenance
11. In case of problems
12. Warranty

01. INTRODUCTION

Thank you for choosing the Babydoppler Connect.

IMPORTANT - PLEASE KEEP THESE INSTRUCTIONS FOR FUTURE USE – READ THEM CAREFULLY

You can connect the Babydoppler Connect to the free Babydoppler Connect application which is available on App Store and Google Play and benefit from the functionalities of our application, hearing and recording the rhythm of your baby's heartbeats and sharing with your loved ones.

02. COPYRIGHT AND TRADEMARK

Cocoon Life is a registered trademark of L.A.T.L. SAS, trading under the name of LATL TECHNOLOGY.

This information leaflet is protected by international copyright. Any reproduction, distribution, translation, or transmission by any means whatsoever of all or part of this document is prohibited save with the prior written consent of LATL TECHNOLOGY.

Apple Store is a registered trademark of APPLE Inc.

Google Play is a registered trademark of GOOGLE Inc.

03. CONTENTS OF THE BOX

- 1 Babydoppler Connect
- 1 double jack cable
- 1 set of headphones
- 1 set of user guide
- 1 ultrasound gel in 40ml

04. MANUFACTURER'S WARNING AND LIABILITY

The Babydoppler Connect has been developed in compliance with the European directive 93/42/EEC (Class I) and associated standards. This doppler is intended for members of the general public over the age of 14 for the purpose of detecting the foetal heartbeat and may not be used to draw up a medical diagnosis or course of therapy based on information gathered through their use.

Only doctors are authorised to issue a medical diagnosis.

If you do not hear the baby's heartbeat, it may be because the doppler is wrongly positioned,

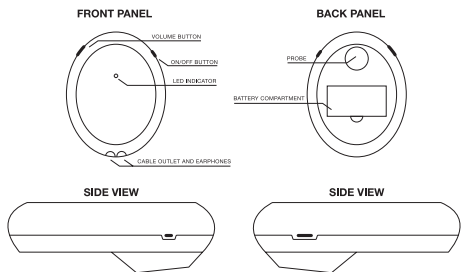
there is no contact gel, the cables are not plugged in properly or the doppler or batteries may be malfunctioning. IN CASE OF DOUBT, CONSULT YOUR MIDWIFE OR GYNAECOLOGIST. The Babydoppler Connect is supplied with a cable specifically developed to allow you to listen to your baby's heartbeat on your smartphone and tablet. THE USE OF ANY CABLE OTHER THAN THE ONE SUPPLIED WITH YOUR BABYDOPPLER CONNECT MAY DAMAGE YOUR SMARTPHONE OR TABLET. ONLY USE THE CABLE THAT HAS BEEN SUPPLIED WITH THE BABYDOPPLER CONNECT.

You should also handle the Babydoppler Connect and its cable with care. Do not twist or damage the cable and do not try to dismantle or repair the doppler or cable yourself.

The information contained in this document is subject to change without notice.

BABYMOOV and LATL TECHNOLOGY declines any liability in case of damage to your doppler, smartphone or tablet linked to the use of a cable other than the one supplied with your Babydoppler Connect or in case of inappropriate use, poor maintenance or changes or repairs made to the doppler and cables by persons not authorised by BABYMOOV and LATL TECHNOLOGY.

05. PRESENTATION OF THE DEVICE

06. PRECAUTIONS FOR USE

The user is an intended operator. The patient can use and maintain the device and its accessories according to this manual. The Babydoppler Connect is powered by two 1,5V AAA-type batteries. To avoid any potential danger, use the Babydoppler Connect in accordance with the following safety rules:

1. Do not open the battery compartment or operating this equipment when the device is switched on.
2. Do not try to open or dismantle it yourself
3. Maintain correct battery polarity (they should face the correct direction)
4. If the doppler is not used for a long period, remove the battery and store it in a cool, dry place
5. Keep your doppler and accessories dry. Humidity and liquids may damage the electronic parts or circuits
6. Do not store your device and its accessories in places with high concentrations of dust. Dust or microparticles may cause malfunctions.
7. Do not keep your device and its accessories near hot areas. The battery may leak.
8. Your doppler is not appropriate for use in the presence of inflammable liquids and gas.

9. Do not throw the battery in a fire, it may explode.
10. Electromagnetic interference – Do not use your doppler in the immediate vicinity of your mobile phone or any radio transmitter; this could cause electromagnetic interference.
11. This device cannot be used with a defibrillator or any high-frequency medical device.
12. Avoid shocks and knocks to the device.
13. Only use your doppler and its accessories for the functions it is intended for.
14. Do not let children use your device and its accessories.
15. The doppler should be applied to healthy skin. If you are not sure of the physiological state of the skin, do not use it.
16. The recommended gel is a hypoallergenic product, non-irritating and non-corrosive. If you make a skin reaction with the gel, stop the application and consult a physician.
17. The doppler should be cleaned after each use according to recommendations of « Maintenance » section.
18. The doppler should not be used before the 12th week of pregnancy.
19. At the end of its life cycle, the doppler and its accessories should be recycled in accordance with the provisions in force.
20. Be careful to strangulation due to cables, particularly due to excessive length.

07. TECHNICAL FEATURES

Trademark	Babydoppler Connect
Commercial reference	Babydoppler Connect
Safety Class	Type CF applied part IEC 60601-1:2005 + A1:2012
Electromagnetic compatibility (EMC)	IEC 60601-1-2 Class B group 1
Internal power supply	This device operates on 3 V DC (2 x 1.5V AAA size batteries) Power consumption < 1 W
Appropriate use	As from the 12 th week of pregnancy
Environment	Working <ul style="list-style-type: none"> •Temperature : between +5°C and 40°C •Humidity ≤ 80% •Atmospheric pressure: between 70 kPa and 106 kPa Transport and storage <ul style="list-style-type: none"> •Temperature: between -10°C and 55°C •Humidity < 93% •Atmospheric pressure: between 50 kPa à 106 kPa
Physical features	Dimensions : 105 x 90 x 36 mm Mass : around 80 g (including the batteries)
Probe	Nominal frequency : 3 MHz Operating frequency : 3 MHz ± 10% Output acoustic power : < 1 MPa $I_{spta} : < 100 \text{ mW/cm}^2$ MI et TI < 1 Output intensity of ultrasound : $I_{ob} < 20 \text{ mW/cm}^2$ Effective Radiating Area of Transducer < 208mm ²

The Babydoppler Connect is very compact, low-consumption portable electronic device intended for self-detection and listening to the foetal heartbeat by pregnant women and their family. Although it is compliant with the European directive 93/42/EEC (Class I) and associated standards, it is not intended for medical diagnostic use and it is used manually in association with the Babydoppler Connect mobile application and the special cable supplied for that purpose. The Babydoppler Connect has a highly sensitive 3 MHz probe and one audio outlet for connection to the smartphone/tablet compatible cable specially designed by BABYMOOV and LATL TECHNOLOGY and supplied with the Babydoppler Connect.

THE USE OF ANY CABLE OTHER THAN THE ONE SUPPLIED WITH YOUR BABYDOPPLER CONNECT MAY DAMAGE YOUR SMARTPHONE OR TABLET. ONLY USE THE CABLE SUPPLIED WITH THE BABYDOPPLER CONNECT.

The intensity of the ultrasounds emitted by the Babydoppler Connect is much weaker than the limit imposed by the regulations in force.

08. SYMBOLS

Logo	Signification
	CE marking
	Catalogue reference
	Serial number
	Consult the documentation
	Type CF : applied part
	Do not throw in trash : Waste Electrical and Electronic Equipment (WEEE)
	Smartphone socket
	Headphone socket
ON/OFF	Power ON/OFF
	Volume adjust
	Date of manufacture
	Manufacturer address
IP	Protection sign

09. USAGE

1. Install the Babydoppler Connect application on your tablet

or smartphone from App Store or Google Play platforms (key word « Babydoppler Connect » in the search bar).

2. Before using the device for the first time, insert the two 1,5V AAA-type batteries as indicated by the polarity markings in the battery compartment of the Babydoppler Connect.

3. Plug the headphones delivered, in the associated jack plug (1)

4. Switch on the Babydoppler Connect by pushing 2 seconds the on/off button on the right side of the device. The power LED indicator light should be lit. If not, please check the polarity of the battery. Light will blink when battery is low.

5. Apply the contact gel (2) and position the doppler probe on the lower abdomen of the mother-to-be according to the diagram and the number of weeks of pregnancy. (3) Move slowly upwards until you hear the baby's heartbeats in your headphones.

6. Adjust the volume (4 levels) of headphones thanks to the button on the left side of the device. Short press volume button, volume change one level. Long press volume button, volume change continuous.

7. Launch the Babydoppler Connect application, register from the homepage.

8. To use the Babydoppler Connect with the application, plug the double jack cable delivered, in the associated jack plug (4) and plug the other end to the jack plug of your tablet/smartphone. Then plug the headphones delivered, in the other jack plug (4).

Finally, on the application press the «Record your baby» function. For a better listening, Airplane Mode activation is recommended.

9. Press the «register» function to save your baby's heartbeats in the history of your plays.

10. To share your plays with your relations, go on «Your recordings» function. Click on the play you want to share, then click on the «share» function. Choose the sending channel (mail, MMS, Social network). From this interface, you are also able to listen your recording by pressing the «listen» function.

10. MAINTENANCE

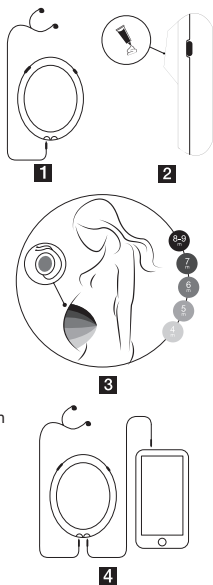
Before cleaning, switch off the device and remove the batteries.

Clean the surface of the device and wipe the probe to remove any traces of contact gel with a soft, dry cloth. If necessary, clean with a soft cloth soaked in soapy water or water and wipe dry immediately with a clean cloth.

Do not use a strong solvent like acetone or an abrasive like steel wool or a metal polisher.

Do not allow the liquid to penetrate inside the device and do not immerse any part of the device in any liquid whatsoever.

To disinfect and sterilise the surface of the probe, use Rubbing Alcohol 70% and leave to dry,

or clean with a soft dry cloth.

Your Babydoppler Connect device is fragile and should be handled with care. The gel must be removed after each use of the doppler. These precautions will prolong the life of your doppler. The user must also make sure before handling it that the doppler does not present any obvious damage which might affect its safety and the correct functioning of the device.

The typical operation time of the batteries is up to 20 days. This equipment expected service life is 10 years. Before operating this equipment make sure to wait 10 minutes in order to let it warm or cool and reach ambient temperature.

It is recommended to check the proper functioning of the device at monthly intervals. If there is any proven damage, it is advisable to replace the doppler before use. Carry out regular safety tests to make sure that the device is insulated against leakage current, including measuring the leakage current. The recommended frequency is once every two years or according to the directives of the country in which the device is used.

The device and accessories are to be disposed of according to local regulations after their service lives. They can be returned to the dealer or the manufacturer for recycling or proper disposal.

The Babydoppler Connect and its mobile application do not provide precise data on the foetal cardiac rhythm but simply a sound recording. If the foetal cardiac rhythm is not credible, there are other methods such as the stethoscope to check it. **Only the doctor is authorised to carry out this kind of measure.**

11. IN CASE OF PROBLEMS

Generally speaking, in case of any problems switch off your doppler, remove the batteries and unplug the cables supplied. Plug the cables back into your smartphone or tablet and your doppler, re-install the batteries and switch your doppler back on.

If the following problems arise again when using your doppler, try to solve them by using the proposed solutions.

Problems	Possible reasons	Solutions
The sound is weak	The volume is too low. The batterie is worn out. You did not apply contact gel.	Adjust the sound volume of the doppler. Change the batteries. Apply gel to the probe.
There is noise	The volume is too high. The power is low. Disturbance from an external signal.	Adjust the sound volume of the doppler. Change the batteries. Make sure your smartphone or tablet is in Airplane Mode.
There is low sensitivity	The probe is wrongly positioned. You did not apply contact gel.	Adjust the position of the probe. Apply gel to the probe.
You cannot hear or record the baby's heartbeat	The position of the probe is incorrect. You did not apply contact gel. The cables are not connected correctly.	Adjust the position of the probe. Apply gel to the probe. Make sure that the cables are correctly connected.

12. WARRANTY

Babydoppler Connect application was developed for the new generation of smartphone and tablets and new versions of iOS/Android operating systems. BABYMOOV and LATL TECHNO-

LOGY cannot guarantee excellent functioning of Babydoppler Connect application for all the smartphone/tablets available on the market and previous versions of iOS/Android operating systems.

In any case, Babydoppler Connect may be used alone (not connected to a smartphone/tablet) with the supplied earphones just to listen the foetal heartbeat.

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