Mumpulse

EN [Heating – Cooling lactation massager](#)

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INTENDED USE

The Mumpülse is designed to improve the comfort of mothers during breastfeeding.

CHARGING YOUR MASSAGER

1. Connect the charging cable provided to the massager
2. Plug the USB into a computer or USB adaptor (DC 5V 1.5A).
3. The indicator will be blinking white while the massager is charging.
4. The massager is fully charged when the indicator remains solid white.
5. It takes approximately 1h30 to get the device fully charged. Recharge the massager as needed.

HOW TO USE MUMPÜLSE ?

VIBRATIONS

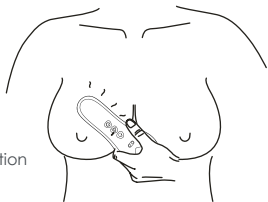
The vibration has a massaging effect that will improve local circulation and can be used before a breast pumping session.

Massage the breast with the curved side, moving towards the nipple or in circular movements from the base to the nipple.


HEATING :

Allow a few minutes for the massager to reach full heat, then warm the breast to bring a feeling of relaxation just before a breast pumping session. Position the heating surface around the areola, on tense areas or on the base of the breast and hold for 5 to 10 minutes.

COOLING :

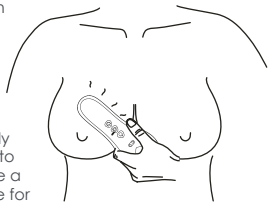Allow a few minutes for the massager to reach full cold, then position the cooling surface to bring to the zone comfort and well-being. You can apply the cooling surface globally to the whole breast or to a specific area, but use a moderate temperature for the area around the areola, which is more sensitive.

NOTE

1. In heating and cooling mode, the device will change from high heat to low heat after 20 minutes. It will shut off automatically after heating for 30 minutes.
2. It cannot be turned on when charging.
3. Fully charge the battery before the first use.

CARE AND STORAGE

- Always clean your massager before and after every use with a soft damp cloth.
- Unplug from power source before cleaning.
- Clean with warm water.
- Do NOT submerge the massager in water.
- Allow the massager to air dry.
- Avoid direct sunlight.
- Store the massager in a clean and dust-free place.

WARNING

- This lactation massager is intended for personal use only. Do not rent or share the product.
- Do not allow anyone unfamiliar with the product to use it.
- Do not use the device if the charging cable or the unit itself is damaged.
- If the cable shows signs of wear (loose casing, exposed wires, etc.), stop using it immediately and contact the manufacturer.
- Only use the charging cable provided with the device.
- Do not clean or sterilize the product in a washing machine, microwave, or dishwasher.
- This vibrating lactation massager is designed to relieve certain breastfeeding-related discomforts.

Use it only as directed in the user manual.

- Parental or guardian supervision is required if the user has physical or intellectual disabilities.
- Do not use the device while breastfeeding or when the baby is in contact with the breast.
- Do not immerse the device in water or any other liquid.
- Discontinue use if you experience pain, significant discomfort, or skin irritation. Consult a healthcare professional.
- Do not use the device while sleeping.
- This product is not a substitute for medical advice. If you are experiencing persistent pain, engorgement, or other breastfeeding issues, consult a qualified healthcare provider or lactation consultant.

PRODUCT SPECIFICATION	
Name and Model	Mumpülse – B0229
Product size	129.9mm x 48.8mm x 42.7mm
Power Supply	DC3.7V / 1100mAh
Charging Port	USB-Magnetic

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

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